

Microsclerotherapy Needle Set 30g

STD Pharmaceutical



**FOR THE TREATMENT
OF TELANGIECTASIAS
AND VENOUS FLARES
BY SCLEROTHERAPY.**



- 30G needle mounted on flexible tubing.
- Minimal movement of needle during aspiration and injection.
- Extra sharp 30G needle retains edge during multiple injections in single session of treatment.
- Needle stability within vein irrespective of syringe plunger movement.



- Angle of needle insertion is not restricted by attachment to syringe, thus making for more accurate venupuncture.
- Fine bore translucent tubing permits observation of minute fleck of blood on aspiration.
- Sterilised by gamma radiation
- Luer fitting.

email: enquiries@stdpharm.co.uk
web page: <http://www.stdpharm.co.uk>

STD Pharmaceutical Products Ltd
Plough Lane
Hereford HR4 0EL, England
Telephone UK (0) 1432 373555
Telefax UK (0) 1432 371314

STD is a registered trade mark
Copyright © STD Pharmaceutical Products Ltd 1995

STD
Pharmaceutical

LIT.9000 AUG 11

Microsclerotherapy Needle Set

Recommended Technique

- Use a filling needle to prepare a 2ml or 5ml syringe with Fibro-vein 0.2%.
- Attach the Microsclerotherapy Needle Set and expel air from the tube.
- Take syringe in left hand (assuming right handed operator) to permit thumb operation on piston.
- Using thumb and index finger of right hand, hold the tubing just behind junction of the needle and the tubing.
- Insert needle into venule, (feeder vein if apparent) aspirate slightly, using left thumb and index finger to show a fleck of blood in the tubing.
- Inject 0.1 - 0.5ml of Fibro-vein 0.2% in to the vein until blanching ceases to spread.
- Inject other feeder veins in the associated area.
- Apply immediate compression to the whole area using a gauze swab and sufficient STD compression pads (cut to size), to cover the area, place pads flat side down.
- Secure with a single STD compression bandage (limited stretch).
- Apply a class 1 compression stocking (Medi recommended).
- Instruct patient to maintain uninterrupted compression for 2 to 7 days depending on the venule size.
- The STD waterproof overstocking permits bathing and showering during this period.